

TOMORROW

non invasive prenatal test


BLOOD SAMPLING INSTRUCTIONS

Before blood sampling, please read carefully the following instructions.

PROCEDURE:

1. Collect the blood sample into a **Streck** tube (ref.: Cell-Free DNA BCT® CE).
2. Identify the Streck Tube with **at least 2 identifications**, for example, patient's name and date of birth;
3. Clean the region of sampling and harvest by venipuncture between **7–10 mL** of blood into the Streck tube;
4. Immediately after sampling, carefully rotate the tube 8-10 times (inadequate or late mixture may affect the test result);
5. Sample should be stored and shipped at room temperature (20-25°C, 68-77°F);
6. Make sure delivery arrives **within 4 days after blood sampling** and that is addressed to:

*CGC Genetics
Rua Sá da Bandeira, 706-1º
4000-432 Porto, Portugal*

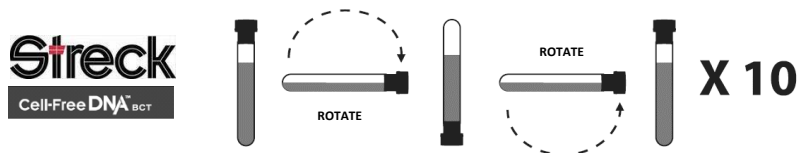

Attention, the test cannot be performed if:

- Blood sample volume is below 7 mL;
- The tube is not identified appropriately;
- The Requisition Form **TOMORROW** is not properly completed and signed by the health professional;
- Patient Informed Consent is not signed by the patient.

- To find answers to the most frequent questions, please visit www.tomorrowtest.com
- For additional information, please send us an email to customer@cgcgenetics.com or call + 351 223 389 900, from Monday to Friday 09:00-18:00 (GMT/UTC + 00:00).